

# **Pfizer Bivalent**

**Reliance Yangzhou Medline Medical Safety MOA CNS 25g 1ml**

**Morbidly Obese CNS Safety Device**

# Box Presentation



# Device Presentation



# Using the Device



The Orange  
Safety Cap  
Is Pulled  
back to  
expose the  
Needle  
contained  
within the  
protective  
sheath




# Using the Device



1. Remove the safety Sheath to allow you to draw up the vaccine
2. Draw up the required dose of vaccine as per training video
3. Administer the Vaccine



# Disposing of Used Device

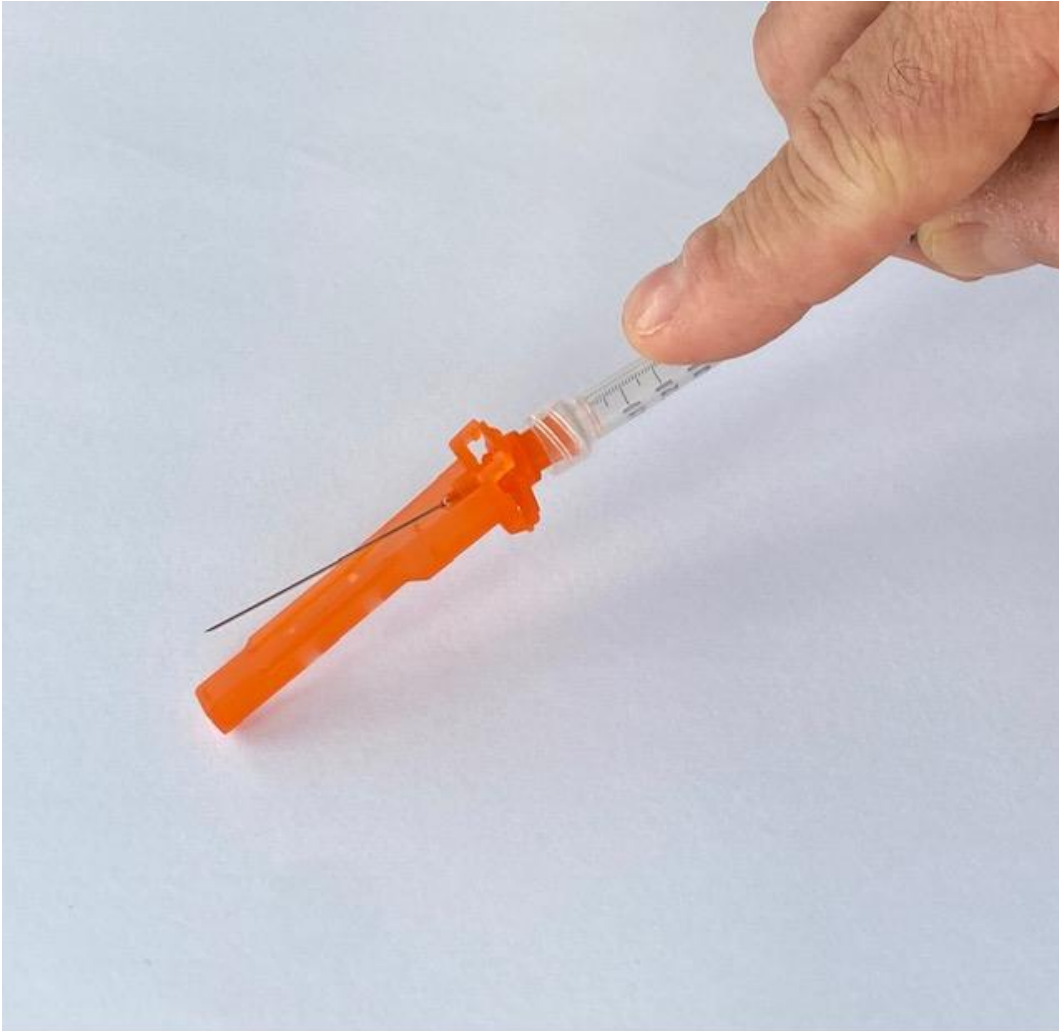

- The protective cap folds back over the needle
- Press it on a firm surface until you hear it lock into place and the needle is covered and secure in the protective cap
- Dispose of Safely in Sharps Bin

